



picopoly®

Dual component material on polyurethane basis – for highly precise models

- Low viscosity
- dimensionally stable
- can be grinded and sawed easily
- very exact
- stable edges

Item no.	Item	Contents
1280 0001	picopoly®-Set (basic colour white)	Set consists of: 3 x 220 g basic 1 x 100 g hardener 3 x measuring cups small 1 x measuring cup large 4 x scoops small 2 x scoops large 1 x colour paste apricot
1280 0002	picopoly®-Set (basic colour white)	Contents like 1280 0001 colour paste golden brown
1280 0003	picopoly®-Set (basic colour white)	Contents like 1280 0001 colour paste platinum-grey

Item no.	Item	Contents
1280 0008	picopoly®-Farbpaste apricot	8 g tube
1280 0018	picopoly®-Farbpaste golden brown	8 g tube
1280 0028	picopoly®-Farbpaste platinum-grey	8 g tube

pico-scanspray

For all CAD/CAM-systems

Provides an exact optical scan quality.
Thinnest coating is sufficient. Easy to remove.

Item no.	Item	Contents
1190 0075	pico-scanspray	75 ml aerosol can

Important:

For applications with polyether moulding, use Majesthetik-Separator (page 27).



picodent® easyflow

Wetting agent

For all impression materials.
For bubble-free models.

Spray on impressions from 20 cm distance. Shake off surplus wetting agent. Do not blow dry.

Item no.	Item	Contents
1503 0250	picodent® easyflow	250 ml pump bottle
1503 1000	picodent® easyflow	1.000 ml bottle
1503 3000	picodent® easyflow	3.000 ml canister



picodent® sockelsep

Separates plaster from plaster

Does not form a film or layer. Easy separation – ideal when using low viscosity base plasters.

Spray on evenly, leave for a while.
Blow surplus off carefully.

Item no.	Item	Contents
1501 0250	picodent® sockelsep	250 ml pump bottle
1501 2000	picodent® sockelsep	1.000 ml bottle
1501 5001	picodent® sockelsep	5.000 ml canister



picosep

Separates plaster from plaster

Can be brushed or sprayed on. Protects plaster models against water, e.g., when trimming.

Apply separator thinly with a brush or spray on generously from approx. 20 cm distance, leave for a while. Carefully blow off excess material. You can apply the plaster after approx. 2-3 minutes.

Item no.	Item	Contents
1501 1250	picosep	250 ml pump bottle
1501 1001	picosep	1.000 ml bottle
1501 5000	picosep	5.000 ml canister



picodent® isoliermittel Gips gegen Kunststoff

Based on alginate, separates plaster from plastic
Well-suited for plastic prosthetics.

Use a brush to apply the material on the lukewarm model and allow a reaction time of 5 minutes. Then repeat the process. Wipe away surplus material. Attention: Always transfer the liquid from the original bottle into a working bottle. Never put back unused material into the original bottle. Do not stick the application brush into the original bottle.

Item no.	Item	Contents
1502 1000	picodent® isoliermittel Gips gegen Kunststoff	1.000 ml bottle



picodent® gipslöser

Plaster solvent with powerful effect.
Also suitable for electroforming technique.

Areas of application:

Mixing bowls, dentures, trays, castings
(plaster bonded investment material)

Application:

Put the objects which need cleaning into the undiluted solvent. Using ultrasound and a mild bath (40°-50°C) will speed up the result. Time: approx. 20-80 minutes. 1.000 ml removes 100 g plaster completely.

Item no.	Item	Contents
1190 0001	picodent® gipslöser	1.000 ml bottle
1190 0005	picodent® gipslöser	5.000 ml canister



picodent® löffelreiniger

Liquid concentrate for cleaning
impression trays and instruments

Also suitable for ultrasonic devices.

Spectrum of application:

Cleaning of impression trays, spatulas and instruments.

Application:

picodent® tray-cleaner removes remainders of zincoxideugenol, carboxylate-, phosphate- and glasionomer cement, adhesive varnish and alganite.

Item no.	Item	Contents
1717 0001	picodent® löffelreiniger tray-cleaner (concentrate)	1.000 ml dosage bottle (= approx 25 litres ready-to-use liquid)
1717 0003	picodent® löffelreiniger tray-cleaner (concentrate)	3.000 ml canister



picodent® dentabs

Optimised flocculant material for plaster skins

Prevents pipe blockages. Saves time and money.

picodent® dentabs is a flocculant that is easy to use in tablet form. The floating matter of plasters, quartz and investment material is completely discarded. Only one dentab ensures clear water for a whole week.

Item no.	Item	Contents
1510 0012	picodent® dentabs	12 pieces included basket
1510 0050	picodent® dentabs	25 pieces included basket
1510 0003	Einhängekorb für picodent® dentabs (basket)	1 piece



picodent® softknete

Re-useable fixation putty for articulation purposes.

Also ideal for blocking of plaster models.

Universal soft plasticine for modell marking. Also suited as a fixation assistant for zeiser® model system.

Item no.	Item	Contents
1305 1000	picodent® softknete	1,000 g