DaVinci's Majesthetik® - Separator

D

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GB

Separating agent perfectly suited for spraying or daubing for Majesthetik gum masks against A or C silicones and for isolating polyether castings before casting with polyurethane (e.g. picopoly).

How to use:

Spray the separator thinly and evenly into all areas or daub on thinly. Leave the excess separator to dry and do not work with compressed air. It will dry in around 1 to 2 minutes. The dried separator should have a matt, non-shiny surface.

Shake before use.**Contents: 50 ml**

Contains: Naphta (petroleum)

H225 Highly flammable liquid and vapour.

H315 Causes skin irritation.

H336 May cause drowsiness or dizziness.

H304 May be fatal if swallowed and enters airways.

H411 Toxic to aquatic life with long lasting effects.

P210 Keep away from heat / sparks / open flames / hot surfaces - No smoking.

P241 Use explosion-proof electrical / ventilating / lighting / equipment.

P301+P310: IF SWALLOWED:

Immediately call a POISON CENTER or doctor/physician.

P303+P361+P353: If on skin (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.

P405 Store locked up.

P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

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Dental-Produktions- und Vertriebs-GmbH