1. Informations importantes

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1. Important information

This assembly and user guide is an integral part of the device. It has been specifically written for this device and complies with the state of the art at the time the item was first put on the market. Before using the device, please read the entire assembly and user manual.

2. General safety and warning indications

- The device should not be operated in rooms where there are flammable substances and / or mixes – there is a risk of the flammable substances igniting and causing an explosion.
- Do NOT look directly at the emitted light beam. When using the device, look through the mounted light shield, or wear the goggles provided.
- Store the device in a dry, dust-free place.
- Do not submerge the device or individual parts of the device into water or allow it to come into contact with humidity. This could cause an electric shock.
- Do not drop the device, because this could damage it.
- Before use, check that the device is working and in good condition.
- Do not alter or modify the device. Only use original parts. The manufacturer assumes no liability for damage that results from unauthorised accessories, special accessories and parts other than original repair parts and spare parts.
- The assembly and user guide is an integral part of the device and all users must have access to it at all times when using the device.

3. Transportation

The original packaging of the device provides optimum protection during transportation and for storing the device. The manufacturer assumes no liability for damage caused during transportation caused by inadequate packaging, even during the guarantee period. The device must therefore only be transported in the original packaging. Keep the packaging away from children.

4. Disposal and environmental protection

This device is labelled in accordance with the 2002/96/EC Directive on Waste Electrical and Electronic Equipment – WEEE). This directive lays down the framework conditions for returning and recycling used devices for the entire EU. The symbol on the product or on the accompanying documents indicates that this device may not be treated as household waste. It must be disposed of at a designated collection point for the recycling of electrical and electronic devices.

5. Intended purpose

The pico-light LED polymerisation light is intended for hardening light-curing composites with a wavelength of 400 +/- 10 nm. It is specially designed for all light-curing picodent® products:

- picobello
- picobloc
- picopreci plus
- Majesthetik® Stumpfix®

6. Use other than for the intended purpose

Another use, or a use other than for the intended purpose - in particular use on or in bodily orifices - is deemed to be incorrect use. The manufacturer may not be held liable for any damage resulting from the above. The user alone bears responsibility for incorrect use.

7. Product description

The pico-light LED is a source of light based on modern LED light technology. It is used for curing (polymerisation) photosensitive viscous composites. The casing is made of a high-quality aluminium alloy. The energy is provided by a rechargeable Li-Ion battery. The light is passed through and focussed with a plug-on light guide made from glass. The connector for the battery charger is located on the side of the casing. An eye protection shield must be put on by the user before using the device. The light is switched on with a button on the casing.

7.1. Overview and delivery contents

Assembly

Operation

Recharge

7.2. Technical Information pico-light LED handpiece

Light source:	LED diode (2 core chip 45 x 45 mil)
Wavelength:	400 +/- 10 nm
Light intensity:	400 - 600 lumen
Diameter of the optical tip:	7,9 mm
Length:	23 cm
Weight (incl. battery):	183 g
Battery power:	4200 mAh
Battery voltage:	3,7 V li-ion

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Dental-Produktions- und Vertriebs-GmbH