pico-pattern

Self-curing universal modelling resin

Applications (colour: red)

- Casting of telescopic and conical crowns
- Adhesive bridges (Maryland bridges)
- Casting of individual attachments and bars
- Fixation of splints e.g. of soldered models
- Casting of inlays, onlays, retentions and clasps
- Casting of superstructures

Applications (shade: dentine)

Fabrication of resin dies in tooth shade – prevents the die from showing through zirconia/ceramic veneering copings

Characteristics and benefits

- Easy processing with the brush method
- Extremely fine powder - less monomer necessary
- Very short curing time
- High strength - very easy to grind
- Very dense and smooth surface
- High precision due to low shrinkage
- Burns without leaving residues
- Very good dosing accuracy

Processing tips:

1. Preparation

Pour the appropriate amount of powder and liquid into two separate mixing vessels (e.g. Resimix beakers).

2. Processing

Dip the brush tip into the liquid and then pick up a small amount of powder. Less fluid is required for wetting due to the fine powder of pico-pattern. The resin is moistened on the brush tip and can be transferred precisely to the object. This process is repeated until the model has been constructed step by step. The curing time is approx. 3 min (depending on the ambient temperature). Always apply pico-pattern as small islands and close the gaps on the surface completely as a second step. Always wipe the brush with the powder and liquid briefly on a strip of paper to remove excess monomer. For safe removal of the model apply a small plastic protrusion in the edge region. After curing, the model can be levered off without risk using an instrument. Remove the protrusion prior to investment. Investment, burn-out and casting is performed according to the specifications of the materials used in each case (e.g. with picovest® royal, the newly developed precision investment material).

When fabricating resin dies, take care that there is a very thin layer insulating the inner surfaces of the crowns when they are filled. Retention pins remain fixed in the rapidly expanding material without any problems.

The tooth-shade material prevents showing through and any shade impairment for the veneer coping.

3. Notes

- Insulation should be used as separator (**Majesthetik® Stumpfix Separator**) for work on a gypsum cast. Depending on the surface structure, also insulate the primary components briefly.
- Clean the brush with the liquid after use or, if necessary, during the work process.
- Under no circumstances put powder or liquid residues back into the containers.

Ordering information:

- pico-pattern SET 100 g powder; 100 ml liquid; 1 x pico-pattern brush; 2 x Resimix beakers 5 ml, colour red
- pico-pattern SET 100 g powder; 100 ml liquid; 1 x pico-pattern brush; 2 x Resimix beakers 5 ml, colour dentin

Notes on storage

Do not use the material after the expiry date. Do not store at temperatures above 25° C. Keep away from sunlight. Always seal containers properly after use. Keep away from heat / sparks / open flames / hot surfaces. Do not smoke. Comply with local regulations when disposing of contents/ containers.

Safety warnings

pico-pattern Liquid contains: methyl methacrylate; N,N-Bis(2-hydroxyethyl)-p-toluidine liquid and vapour easily inflammable.

Harmful if inhaled. May irritate the respiratory system. Causes severe eye irritation. Causes skin irritation. May cause allergic skin reactions. Keep away from heat / sparks / open flames / hot surfaces. Do not smoke. Wear protective gloves / protective clothing / protective eyewear / face protection.

IF CONTACT WITH SKIN OCCURS: gently wash with plenty of soap and water.

pico-pattern Pulver contains: dibenzoylperoxide

May cause allergic skin reactions. Do not inhale dust / smoke / gas / mist / vapour / aerosol. Do not wear contaminated working clothes outside the workplace. Wear protective gloves / protective clothing / protective eyewear / face protection. IF CONTACT WITH SKIN OCCURS: wash with plenty of soap and water. In case of skin irritation or a rash: seek medical advice / call in medical assistance. Take off contaminated clothing and wash before wearing again.

Contraindication

Do not use in case of proven allergy to one or more of the product's constituents.

Please also observe the relevant valid safety data sheets!

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