Scantist 3D - vanishing

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Rev. 2023-06-27

GB Scantist 3D - vanishing

Dental Scan Spray for extraoral use

Processing:

1. Application

Apply SCANTIST 3D - vanishing evenly from a distance of 5-10 cm. Spray the entire surface to be scanned. SCANTIST 3D - vanishing is applied "wet" in this case. The solvent evaporates within a few seconds, while the active ingredient remains on the surface as a coating. The whiteness level of the coating increases as the solvent evaporates. If drops form on the scan object or if the applied matting remains "wet" for a longer period, increase the spray distance or increase the spray speed. The ideal ambient temperature is 21°C/69.8°F.

2. Scanning

The object can be scanned as usual after SCANTIST 3D - vanishing has dried completely.

3. Simply wait

The applied layer of SCANTIST 3D - vanishing evaporates by itself after scanning. The otherwise laborious removal of the coating following application is no longer necessary.

Danger: Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames as well as other ignition sources. Do not smoke. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50 °C/122 °F.

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