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Light-Curing Modelling Resin

Product Description

picobello products are light-curing modelling resins for dental applications, especially milling and attachment jobs along with intraoral splinting and transfer of objects. **picobello** patterns can be tried in the mouth before casting or milling. **picobello form** is available with **picobello gel** as a modelling paste for greater material application. The gel can be applied directly from the dispensing tip, while the paste is taken in portions directly from the light-protected container. **picobello** materials burn without residue and can be cured easily with standard light units (320-500 nm). They can be combined with standard waxes after polymerisation.

Composition

Preparation of acrylates and phosphine oxides.

Areas of Application

- Modelling of attachments, telescopic and conical crowns, etc.
- Modelling of implant abutments and bars
- Splinting of cast parts (intra/extraoral)
- Fabrication of fixation and transfer jigs
- Fabrication of checkbites and stops

- Modelling of adhesive bridges, inlays and onlays
- Intraoral fixation of prostheses
- Modelling of indirect post abutments and core build-ups in a direct method
- Modelling of crowns and bridges for pantographic transfer

Characteristics and Benefits

picobello patterns are very stable, precise and can be taken off without delay. Patterns from light-curing and conventional waxes can be perfectly combined and corrected. **picobello gel** flows out of the syringe during application and is nonetheless instantly firm. In other words, it requires little use or time, and placement is exact. **picobello form** is especially suitable for jobs that require a greater application of material (e.g. bars or transfer jigs). **picobello** materials burn without residue and guarantee superb fitting accuracy.

Instructions for Use:

picobello gel

Processing should be carried out at room temperature (20-22 °C).

After removing the sealing cap, you should attach the enclosed dispensing tips for precisely accurate application.

These should only be used once. Reseal the push syringe afterwards.

Build up your temporary casts in layers no thicker than 1 mm. Interim curing is done for 10 seconds with hand-held light units or in one minute in laboratory devices with a light range of 320-500 nm. Polymerising layers of more than 1 mm may result in incomplete curing, excessive release of heat and stresses. The casts may not fit precisely as a result.

picobello form

Take the lid off the can and remove the desired amount of material, using a spatula or another suitable tool.

After removal, always carefully seal the container to prevent premature polymerisation of the material. **picobello form** can be easily modelled by hand thanks to its kneadable consistency.

Important: Only remove the inhibition layer on the surface (on the model respectively outside the mouth with alcohol) after final completion of the work;

it ensures optimum bonding of the layers together and prevents stresses in the modelling resin.

To achieve the best casting results, complete removal of the inhibition layer is essential.

When working with large volumes, a thin wax coating should be added to the temporary cast.

Polymerisation Final Curing

picobello materials can be cured in common light units with a spectrum of 320-500 nm. Curing is done without vacuum. The average polymerisation times are:

- 90 seconds with stroboscopic devices (Dentacolor)
- 20-30 sec in LED devices (pico-light LED)
- 3.5-5 minutes in devices with UV-A lamps (Megalight)

Please note that the curing time is critically dependent on the emission performance of the lamps used. Correct maintenance of the devices and regular replacement of the light fixtures contribute to consistent curing. Also follow the particular device manufacturer's instructions. If using in a patient's mouth, please be aware that the material heats up during curing. To avoid unacceptable temperature peaks, you should therefore build up larger amounts (> 0.5 g) in layers and polymerise.

Warnings

Contains phosphine oxide and acrylates. Avoid any eye- or skin contact with uncured material as well as with the inhibited layer. Wear suitable protective gloves, protective clothing and eye protection. On sensitive persons sensibilisations cannot be excluded. In this case it is recommended to discontinue the use of the material. Do not breathe dust/ vapours. Switch on aspirator when grinding. Uncured material is dangerous for the environment. Avoid release to the environment. Please pay attention to the valid material safety data sheets also.

Contraindications, Interactions

Do not use the product if you have a known allergy to one of its materials. Otherwise, the patient group is not restricted. There are no known Interactions.

Secondary effects

Product may cause allergic reaction.

Storage Instructions

Store at 4 - 25 °C

Keep **picobello** products away from heat and sunlight.

Do not use after the expiry date.

The product was developed for use in the dental field and must be used in accordance with the directions for use. The manufacturer accepts no liability for damage caused by the product being used in other ways. Furthermore, the user is obliged to test the material on his or her own responsibility for suitability and usability for the intended purposes before it is used, especially if this is not specified in the directions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority

Disposal Instructions

Comply with local regulations when disposing of contents/containers

Manufacturer
DeltaMed GmbH
Raiffeisenstr. 8a
61169 Friedberg | Germany
www.deltamed.de

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picobello gel: 3 x 3 g Slide syringes incl. 12 dispensing tips

picobello gel: 5 x 3 g Slide syringes incl. 20 dispensing tips

picobello form: 25 g in light-protected can

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