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pico-strahlresin

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Instructions

pico-strahlresin

Specially for polishing resin prostheses

Thanks to **pico-strahlresin** the same polishing technique can be used as that for working with pumice stone powder.

Instead of using water, it is recommended to use mixing liquid for pumice powder (e. g. **pico-bimsclean**). The mixing liquid for pumice powder is placed in the polishing tray and filled up with **pico-strahlresin** until a pumice mix is formed. It should be possible to pick up the mix without it running through the fingers so that a certain amount of adhesion is ensured for the object to be polished.

Care must be taken to ensure the resin surfaces to be polished are continually moistened with damp **pico-strahlresin** to prevent any overheating. Usual professional brushes can be used for the pre-polish. In most cases, cloth buffing wheels and goat hair brushes are adequate to create a sufficiently strong level of abrasion to remove uneven surfaces and at the same time to produce an extremely smooth surface.

Now, the high-shine polishing process can be carried out as usual.

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Dental-Produktions- und Vertriebs-GmbH