DaVinci's Majesthetik® - Separator

D

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GB

Separating agent perfectly suited for spraying or daubing for Majesthetik gum masks against A or C silicones and for isolating polyether castings before casting with polyurethane (e.g. picopoly).

How to use:

Spray the separator thinly and evenly into all areas or daub on thinly. Leave the excess separator to dry and do not work with compressed air. It will dry in around 1 to 2 minutes. The dried separator should have a matt, non-shiny surface.

Shake before use.

Contents: 50 ml

Contains: Hydrocarbons, C7-C9, n-Alkane, Isoalkane, Cyclohexane

H225 Highly flammable liquid and vapour.

H336 May cause drowsiness or dizziness.

H304 May be fatal if swallowed and enters airways.

H411 Toxic to aquatic life with long lasting effects.

P210 Keep away from heat / sparks / open flames / hot surfaces - No smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P301+P310: IF SWALLOWED:

Immediately call a POISON CENTER or doctor/physician.

P304+P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.

P331 Do NOT induce vomiting.

EUH066 Repeated exposure may cause skin dryness or cracking.

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Dental-Produktions- und Vertriebs-GmbH