pico-pattern

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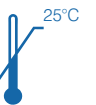
Telefon: +49 2267 6580-0 • www.picodent.de

pico-pattern

Self-curing universal modelling resin

Applications (colour: red)

- Casting of telescopic and conical crowns
- Adhesive bridges (Maryland bridges)
- Casting of individual attachments and bars
- Fixation of splints e.g. of soldered models
- Casting of inlays, onlays, retentions and clasps
- Casting of superstructures

Applications (shade: dentine)

Fabrication of resin dies in tooth shade – prevents the die from showing through zirconia/ceramic veneering copings

Characteristics and benefits

- Easy processing with the brush method
- Extremely fine powder - less monomer necessary
- Very short curing time
- High strength - very easy to grind
- Very dense and smooth surface
- High precision due to low shrinkage
- Burns without leaving residues
- Very good dosing accuracy

Processing tips:

1. Preparation

Pour the appropriate amount of powder and liquid into two separate mixing vessels (e.g. Resimix beakers).

2. Processing

Dip the brush tip into the liquid and then pick up a small amount of powder. Less fluid is required for wetting due to the fine powder of pico-pattern. The resin is moistened on the brush tip and can be transferred precisely to the object. This process is repeated until the model has been constructed step by step. The curing time is approx. 3 min (depending on the ambient temperature). Always apply pico-pattern as small islands and close the gaps on the surface completely as a second step. Always wipe the brush with the powder and liquid briefly on a strip of paper to remove excess monomer. For safe removal of the model apply a small plastic protrusion in the edge region. After curing, the model can be levered off without risk using an instrument. Remove the protrusion prior to investment. Investment, burn-out and casting is performed according to the specifications of the materials used in each case (e.g. with picovest® royal NF, the newly developed precision investment material).

When fabricating resin dies, take care that there is a very thin layer insulating the inner surfaces of the crowns when they are filled. Retention pins remain fixed in the rapidly expanding material without any problems.

The tooth-shade material prevents showing through and any shade impairment for the veneer coping.

3. Notes

- Insulation should be used as separator (**Majesthetik® Stumpfix Separator**) for work on a gypsum cast. Depending on the surface structure, also insulate the primary components briefly.
- Clean the brush with the liquid after use or, if necessary, during the work process.
- Under no circumstances put powder or liquid residues back into the containers.

Ordering information:

- **pico-pattern Set, red:** 100 g powder red; 100 ml liquid; 1 x pico-pattern brush; 2 x Resimix beakers 5 ml
- **pico-pattern Set, dentin:** 100 g powder dentin; 100 ml liquid; 1 x pico-pattern brush; 2 x Resimix beakers 5 ml

Notes on storage and waste treatments

Do not use the material after the expiry date. Do not store at temperatures above 25° C. Always seal containers properly after use. Keep away from heat / sparks / open flames / hot surfaces. Do not smoke. Dispose of contents/container to as hazardous waste - in accordance with local and national legislation - suitable, approved incinerator for combustible organic waste.

Safety warnings

pico-pattern Liquid contains: Methyl methacrylate 7,7,9(or 7,9,9)-trimethyl-4,13-dioxo-3,14-dioxo-5,12-diazahexadecane-1,16-diyl bismethacrylate 1,4-Butandiol dimethacrylate. Highly flammable liquid and vapour. Harmful if swallowed. Causes skin irritation and serious eye irritation. May cause an allergic skin reaction. May cause respiratory irritation. Harmful to aquatic life with long lasting effects. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection/hearing protection. IF ON SKIN: Wash with plenty of Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. In case of fire: Use Foam, Extinguishing powder, Carbon dioxide (CO2) to extinguish. Store in a well-ventilated place. Keep cool.

pico-pattern Pulver contains: dibenzoyl peroxide; benzoyl peroxide. May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing dust/fume/gas/mist/vapours/spray. Contaminated work clothing should not be allowed out of the workplace. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection/hearing protection. IF ON SKIN: Wash with plenty of water. If skin irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse.

Contraindication

Do not use in case of proven allergy to one or more of the product's constituents.

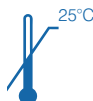
Please also observe the relevant valid safety data sheets!

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