picobello gel | picobello plus | picobello form

Instructions for use en

Product description
picobello products are light-curing, sculpting resins for use in dental technology, in particular for milling and precision attachments and intraoral connection and transfer of objects. **picobello** patterns can be tried in intraorally prior to casting or milling processes.
In addition to **picobello** modelling resins, **picobello form** can be used where the application of larger amounts may be necessary.
The gel can be applied directly from the syringe, the paste can be taken in portions from the light protected container. **picobello** materials burn without residue and can be cured using any standard light-curing unit (320-500 nm). After curing, any standard wax can be combined.

Composition
Composition of acrylic resin and initiators.

- Areas of application**
- Build-up of indirect posts and cores using the direct technique
 - Intraoral fixation of dentures
 - Fabrication of check bites and stops
 - Fabrication of fixation- and insertion guides
 - Connecting of cast units (intra-/extraoral)
 - Contouring precision attachments, telescope and conical crowns etc.
 - Contouring of individual implant abutments and bars
 - Contouring adhesive bridges, inlays and onlays
 - Contouring of crowns and bridges for pantographic transmission

Contraindications:
Patient and user: Products should not be used in patients or by users who are known to be allergic to any of the ingredients. Otherwise the patient group is not restricted.

Adverse reactions:
Patient and user: Product can cause allergic reactions.
User: Observe warnings.

Interactions:
Interactions are not known.

Properties and advantages
picobello patterns are highly stable and precise and can be removed without distortion. Patterns made of light cured or standard waxes can be combined ideally. **picobello gel / picobello plus** is fluid when applied, but is immediately positionally stable, i.e. it can be applied quickly, sparingly and accurately.
picobello form is suitable for works which require the application of larger amounts (e.g. bars or transfer guides).
picobello materials burn without residue and guarantee an excellent fit.

Instructions for use
picobello gel / picobello plus
Processing should be carried out at room temperature (20-22 °C).
After removing the seal cap, attach the application tip supplied for precise application. These are intended for single use. Then seal the slide syringe again.

picobello form
Remove the cap of the plastic container and remove as much material as needed with a spatula or a different suitable instrument. After remove, seal the container thoroughly to prevent the polymerisation of the material ahead of time. **picobello form** can be sculpted and contoured easily by hand due to its kneadable consistency.

Important: Build up the pattern in layers no thicker than 1 mm and cure the layers for 10 seconds with a handheld light-curing unit or for one minute in a laboratory unit with a light range of 320-500 nm. Polymerising layers thicker than 1 mm can result in incomplete curing, excessive thermal release and stresses and do not guarantee an exact fit of the casting. Do not remove the inhibition layer from the surface;
The inhibition layer should only be removed from the surface (on the model or extra-orally using alcohol or acetone) after contouring of the pattern is completed. It provides an optimum bond between the layers and prevents stresses in the modelling resin.
To obtain optimum casting results, complete removing of the inhibition layer is essential.

On large-volume patterns, object should be complemented with a thin wax coating.

- Polymerisation**
picobello materials can be cured in standard light curing units with a spectrum between 320 – 500 nm. Polymerisation is performed without vacuum. The average curing times are:
- 90 sec with strobe lamps (Dentacolor)
 - 20-30 sec in LED devices (pico-light LED)
 - 3.5 – 5 min in devices using UV-A bulbs (Megalight)

Please note that curing times depend on the emission power of the lamps used in light curing devices. Correct maintainace of these devices as well as regular replacement of illuminants account for a constant curing. Please also note the instructions of the device manufacturer.
Please note that the material heats up when used intraorally. To avoid unacceptable temperature peaks, larger amounts (>0,5 g) should be polymerised in layers.

Warnings
Any deviation from the processing process described here can jeopardize the biocompatibility, functionality and safety of the final product. Please always observe the respective valid safety data sheets. Contains acrylates, phosphine oxide. May cause an allergic skin reaction. Avoid contact with unpolymerized material. Wear protective gloves / protective clothing / eye protection. Avoid breathing dust / vapours. Switch on extraction system. The unpolymerized is harmful to the environment. Avoid release to the environment.

Storage
Store at 4-25°C.
Protect **picobello** products from heat and sunlight.
Do not use after the expiry date. Information on shelf life: See label.

Disposal instructions
Dispose of contents/ container in accordance with local regulation.

- Pack contents**
picobello gel: 3 x 3 g in slide syringe, 12 tips
picobello gel: 5 x 3 g in slide syringe, 20 tips
picobello plus: 3 x 3 g in slide syringe, 12 tips
picobello plus: 5 x 3 g in slide syringe, 20 tips
picobello form: 25 g in light protected container

This product was developed for use in dentistry and should be used according to the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority.

	Batch code		Manufacturer
	Catalogue number		Keep away from sunlight
	Use-by date		Exclamation mark
	Consult instructions for use		Environmental hazard
	Temperature limit		Medical Device
	Distributor		

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