picopreci plus

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picopreci plus

Light-Curing Precision Resin

Description
picopreci plus is a light-curing gel-like precision modelling resin for correction and repair of dentures. It makes precisely accurate work possible both when used on models in a dental laboratory and directly in a patient's mouth.
picopreci plus meets the performance characteristics of DIN EN ISO 20795-1 Type 4.

Composition
Composition of acrylates and phosphinate.

- Indication**
- **Fine corrections and repairs in pink resin, e.g.**
Sealing of micro-gaps, temporary casting of interproximal areas for model casting, border moulding of implant workpieces, repair of fractures, spalling, facings, and base parts, reattachment of teeth to prostheses, correction of bubbles, pores, cracks in denture bases.
 - **Fixation of teeth, retentions, anchors or attachments to prostheses (extra/intraoral)**
 - **Application of gingival parts to prostheses, e.g.**
Modelling of papillae, individual characterisation of the gingiva, sealing of screw channels in the mouth.

Intended Purpose
Manufacture, correction and repair of dentures.

Contraindications
Patient and user: Products should not be used if an allergy to one of the constituents is known. Otherwise, the patient group is not restricted.

Adverse effects:
Patient and user: Products may cause allergic reactions.
User: Observe warning notes.

Interactions
No interactions are known.

Characteristics and Benefits
picopreci plus makes fast and precise work possible with short polymerisation times. The material is characterised by precise application directly from the syringe with an application tip and by its high stability, i.e. it requires little use or time, and placement is exact.

Processing instructions
After removing the sealing cap, you should attach the enclosed dispensing tips for precisely accurate application. These should only be used once. Reseal the syringe afterwards. Aim for a consistent layering thickness (about 1-2 mm) during the application.

Curing
Light units with stroboscopic light (e.g. Dentacolor) require 3 minutes, while light units with UV-A tubes (e.g. Megalight) need 10-20 minutes depending on the number of tubes. Bright plaster models (Implant-rock, picodent) assist curing.
Curing with LED lamps is possible in 20 seconds if their emission range is 360-420 nm (e.g. pico-light LED).

The device used must be maintained and calibrated according to the manufacturer's specifications to ensure mechanical properties, color and biocompatibility.
If using in a patient's mouth, please be aware that the material heats up during curing. To avoid unacceptable temperature peaks, you should therefore build up larger amounts (> 0.5 g) in layers and polymerise.
Important: When working on the model, always wet with alginate insulation agent. With application to PMMA plastics for chemical retention, always dab the corresponding areas outside of the mouth with ACRIFIX Bonder beforehand and light-cure (see table). For mechanical retention, the parts can additionally be prepared by being roughened with sandpaper. To be able to apply material after curing, do not remove the inhibition layer on the surface; it ensures optimum bonding and prevents stresses in the modelling resin. If the inhibition layer has already been removed, **picopreci plus** Bonder should be applied outside of the mouth again for reliable bonding.
Only after final completion is the surface released from the inhibition layer and polished with rotating instruments or isopropanol.

Warnings
Any deviation from the described processing procedure may affect the biocompatibility, functionality as well as the safety of the final product. Please always observe the respective valid safety data sheets.

picopreci plus: Contains skin sensitizing substances, acrylates, phosphinate. May cause an allergic skin reaction. Avoid contact with unpolymerized material. Wear protective gloves / protective clothing / eye protection. Avoid breathing dust / vapours. When elaborating out switch on extraction system. If on skin: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice / attention. The unpolymerized material is harmful to the environment. Avoid release to the environment.

Additionally for **picopreci plus** Bonder: Contains skin sensitizing substances: MMA, PETIA, PETTA, phosphinate. Highly flammable liquid and vapour. Causes skin irritation. Causes serious eye damage. May cause respiratory irritation. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Always observe the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

Storage instructions / shelf life:
Protect from exposure to light. Store between 4°C and 25°C (39°F – 77°F). Keep container tightly closed. Do not use after the expiration date. See imprint/label for expiration date.

Disposal instructions
Dispose content/container in accordance with local disposal regulations.

Delivery Unit
picopreci plus Set: 3 x 3 g **picopreci plus** colour: **pink** in light-protected Slide syringes incl. 12 dispensing tips, 5 ml **picopreci plus** Bonder
picopreci plus Set: 5 x 3 g **picopreci plus** colour: **pink** in light-protected Slide syringes incl. 20 dispensing tips
picopreci plus Set: 3 x 3 g **picopreci plus** colour: **opaque pink (34)** in light-protected Slide syringes incl. 12 dispensing tips, 5 ml **picopreci plus** Bonder
picopreci plus Set: 3 x 3 g **picopreci plus** colour: **tooth-coloured** in light-protected Slide syringes incl. 12 dispensing tips, 5 ml **picopreci plus** Bonder

The product was developed for use in the dental field and must be used in accordance with the directions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority.

	Batch number		Distributor
	Item number		Keep away from light
	Shelf life		Exclamation mark
	Observe instruction for use		Environmental hazard
	Temperature limit		Caustic effect
	Medical device		Flame
	Manufacturer		

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picodent Dental-Produktions- und Vertriebs GmbH